

Available on FP10/GP10 – PLEASE COMPLETE ALL SECTIONS
TO BE COMPLETED BY THE HEALTHCARE PROFESSIONAL AND GIVEN TO YOUR PRESCRIBER

1 Order No.
Patient Name
Altiform Registration No. (Repeat Orders)

DELIVER TO PHARMACY/HOSPITAL:
Name
Address
..... Postcode
Telephone No.

MEASUREMENTS COMPLETED BY:
Health Centre
.....
Contact No.

2 INVOICE TO:

Hospital/Clinic
 Buying Group
 Wholesaler
 Pharmacy Chain

Name of buying group

Account No.

3 COMPRESSION CLASS:
 1 (14-17 mmHg) 2 (18-24 mmHg) 3 (25-35 mmHg)

STYLE:
 Below Knee Thigh Length Hold Ups:
 Thigh Length Standard Plain Lace

FOOT:
 Closed Toe Open Toe

If you have chosen thigh length stockings, choose what styles you would like, at no extra cost.

4 COLOUR: Beige Black


QUANTITY (SINGLES): Left Leg Right Leg

5 MEASUREMENTS:

MEASUREMENT TIPS:

- Measurements should be taken with the person seated, feet flat on the floor.
- For foot measurement: stand on a plain sheet of paper, mark at the toe and at the heel; remove the foot and measure between the points; this will provide a more accurate reading.
- When possible, measurements should be taken first thing in the morning. ⁽¹⁾
- For thigh length, take the measurement while the person is standing.

1. NICE Clinical Knowledge Summaries

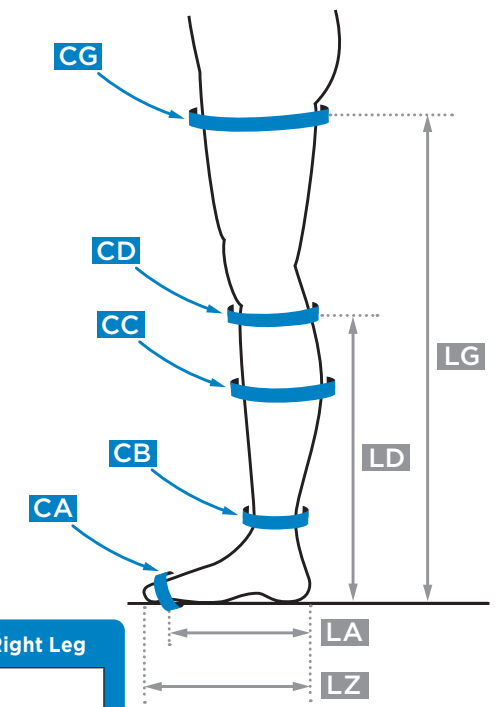

Please fill in all the appropriate measurements in centimetres

CIRCUMFERENCE MEASUREMENTS	Left Leg	Right Leg
CG (thigh - thigh length only)		
CD (under knee)		
CC (calf)		
CB (ankle)		
CA (measure at base of little toe)		

LENGTH MEASUREMENTS	Left Leg	Right Leg
LG (for thigh length only)		
LD		
LA (for open toe only)		
LZ (for closed toe only)		

COMMENTS:

Available on FP10/GP10 - PLEASE COMPLETE ALL SECTIONS
TO BE COMPLETED BY THE HEALTHCARE PROFESSIONAL AND GIVEN TO YOUR PRESCRIBER

1 Patient Name
Doctor/Clinician Clinic
.....
Pharmacist
Date

2 AVAILABLE STYLES:

	BELOW KNEE		THIGH LENGTH	
	Open Toe	Closed Toe	Open Toe	Closed Toe
Class I	Beige	Beige or Black	Beige	Beige or Black
Class II	Beige	Beige or Black	Beige	Beige or Black
Class III	Beige	Beige	Beige	Beige

3 COMPRESSION CLASS:

1 (14-17 mmHg) 2 (18-24 mmHg) 3 (25-35 mmHg)

STYLE:

Below Knee Thigh Length

FOOT:

Closed Toe Open Toe

4 QUANTITY (PAIRS): Beige Black
(Class I & II Closed Toe only)

5 MEASUREMENTS:

MEASUREMENT TIPS:


- Measurements should be taken with the person seated, feet flat on the floor.
- For foot measurement: stand on a plain sheet of paper, mark at the toe and at the heel; remove the foot and measure between the points; this will provide a more accurate reading.
- When possible, measurements should be taken first thing in the morning. ⁽¹⁾
- For thigh length, take the measurement while the person is standing.

1. NICE Clinical Knowledge Summaries

All measurements are in centimetres

Mid thigh (A) measurement is only required for thigh length stockings

Foot (D) measurement is only required for closed toe stockings

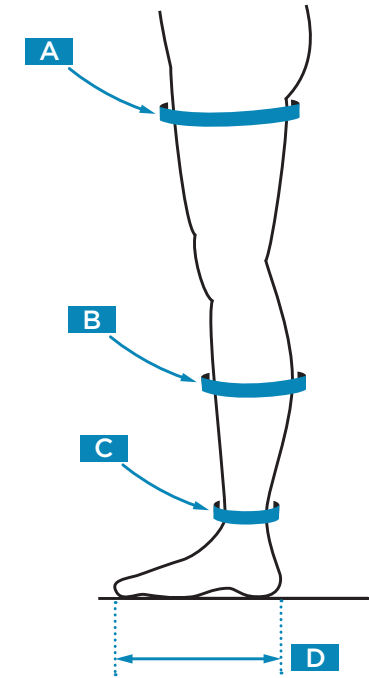

	A: Mid Thigh	B: Calf	C: Ankle	D: Foot	Size Required
Small	41 - 54	32 - 38	19 - 25.5	20 - 26	
Medium	44 - 58	34 - 40.5	21 - 27.5	22 - 28	
Large	47 - 61	36 - 43.5	23 - 29.5	24 - 31	
X Large	50 - 64	38 - 47	25 - 32	26 - 33	

COMMENTS:

NEED HELP? Contact Urgo Medical for additional training and CPD modules **T: 0330 128 0898**

Available on FP10/GP10 - PLEASE COMPLETE ALL SECTIONS
TO BE COMPLETED BY THE HEALTHCARE PROFESSIONAL AND GIVEN TO YOUR PRESCRIBER

1 Order No.

Patient Name

Altipress Registration No. (Repeat Orders)
.....

DELIVER TO PHARMACY/HOSPITAL:

Name

Address

Postcode

Telephone No.

MEASUREMENTS COMPLETED BY:

Health Centre

Contact No.

2 INVOICE TO:

- Hospital/Clinic
- Buying Group
- Wholesaler
- Pharmacy Chain

Name of buying group
.....

Account No.
.....

3 KITS:

Kits contain 2 x white liners (10 mmHg each) and 1 x outer stocking (30 mmHg) in either Black or Soft Beige.

COLOUR:

- Soft Beige
- Black

QUANTITY (KITS):

Left Leg Right Leg

LINER PACKS:

Liner packs contain 2 x white liners (10 mmHg each).

QUANTITY (LINERS):


Left Leg Right Leg

4 MEASUREMENTS:

MEASUREMENT TIPS:

- Measurements should be taken with the person seated, feet flat on the floor.
- For foot measurement: stand on a plain sheet of paper, mark at the toe and at the heel; remove the foot and measure between the points; this will provide a more accurate reading.
- When possible, measurements should be taken first thing in the morning. ⁽¹⁾

1. NICE Clinical Knowledge Summaries

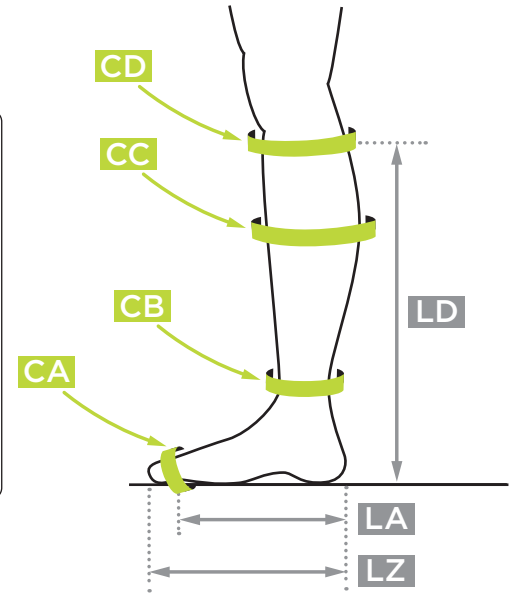

Please fill in all the measurements in centimetres

CIRCUMFERENCE MEASUREMENTS	Left Leg	Right Leg
CD (under knee)		
CC (calf)		
CB (ankle)		
CA (measure at base of little toe)		

LENGTH MEASUREMENTS	Left Leg	Right Leg
LD (below knees)		
LA (base of toes to heel)		
LZ (toe to heel)		

COMMENTS:

Available on FP10/GP10 - PLEASE COMPLETE ALL SECTIONS
TO BE COMPLETED BY THE HEALTHCARE PROFESSIONAL AND GIVEN TO YOUR PRESCRIBER

1 Patient Name

Doctor/Clinician Clinic

.....

Pharmacist

Date

2 KITS:

Contain 2 x White Closed Toe Liners (10 mmHg each) and
1 x Outer Open Toe Stocking (30 mmHg) in Soft Beige

QUANTITY:

LINER PACKS:

Contain 3 x Liners (10 mmHg each)
Only available in White, Soft Beige and Closed Toe

COLOUR:

Soft Beige White


QUANTITY:

3 MEASUREMENTS:

MEASUREMENT TIPS:

- Measurements should be taken with the person seated, feet flat on the floor.
- For foot measurement: stand on a plain sheet of paper, mark at the toe and at the heel; remove the foot and measure between the points; this will provide a more accurate reading.
- When possible, measurements should be taken first thing in the morning. ⁽¹⁾

1. NICE Clinical Knowledge Summaries

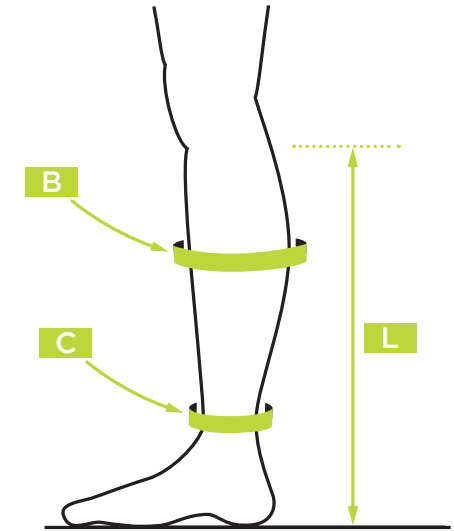

All measurements are in centimetres

	B: Calf	C: Ankle	Size Required
Small	32 - 38	19 - 25.5	
Medium	34 - 40	21 - 27.5	
Large	36 - 42	23 - 29.5	
X Large	38 - 45	25 - 31.5	
XX Large	40.5 - 50	26.5 - 34	

	L: Leg	Size Required (please tick)
Short	34 - 38	<input type="checkbox"/>
Regular	38 - 42	<input type="checkbox"/>
Long	42 - 46	<input type="checkbox"/>

COMMENTS:

.....

**COMPLETE
COMPRESSION
SOLUTIONS**





Urgo Medical, a market leader in advanced wound care and compression, provides a complete range of compression solutions:

UrgoKTwo
DUAL
COMPRESSION SYSTEM

Altipress[®]
LEG ULCER KITS
AND LINER PACKS

Altiform[®]
BRITISH STANDARD
COMPRESSION HOSIERY

Altiven[®]
EUROPEAN STANDARD
COMPRESSION HOSIERY

NEED TRAINING OR SUPPORT?

As part of our commitment to education & appropriate prescribing, Urgo have a clinical team available to train, educate & support formulary or pathway implementation

- Study days, including CPD modules, practical sessions & product demonstrations
- Online resources for pharmacists
- Support developing and evaluating pathways

For more information go to **www.urgo.co.uk**

Urgo Limited
Sullington Road, Shepshed,
Loughborough, LE12 9JG

Contact:

Name:.....

Tel.:.....


DUAL COMPRESSION SYSTEM

Dual compression system that guarantees reliable continuous compression, regardless of patient mobility. UrgoKTwo donates an average pressure of 40mmHg¹ at the ankle.



KTech: short-stretch base layer that delivers:

- 80% of compression
- Stiffness
- Maximum comfort
- Effective absorbency

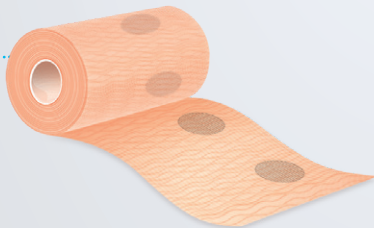

KPress: long-stretch top layer that:

- Provides additional 20% of compression, in order to achieve therapeutic pressure
- Holds system in place

We recommend that **UrgoKTwo** is used for the treatment of all venous and mixed aetiology leg ulcers, regardless of patient mobility.

*We recommend that during **UrgoKTwo** treatment, **Altiform** or **Altiven** is applied as prevention on the non-ulcerated leg. Patients with healed ulcers should also wear preventative hosiery.*

INDICATIONS & KIT SELECTION

Select your **UrgoKTwo** kit based on these 3 criteria:

1 - LEVEL OF COMPRESSION



UrgoKTwo
Full
Compression



UrgoKTwo
Reduced
Compression

2 - PATIENT ANKLE CIRCUMFERENCE



3 - REGULAR OR LATEX FREE

UrgoKTwo

UrgoKTwo
Latex free

For patients or caregivers allergic to Latex

LEG ULCER KIT

The **Altipress 40 Leg Ulcer hosiery kit** is a discreet alternative to bandage systems.

The 2 layers of the Altipress 40 kit provide 40 mmHg pressure at the ankle when used together.

- **LAYER 1:** Liner (10 mmHg at the ankle)
- **LAYER 2:** British Standard Class III stocking (30 mmHg at the ankle)

All Altipress products are latex-free.

AVAILABLE IN:

STANDARD SIZES

Sizes: S, M, L, XL, XXL.

Lengths: Short, Regular, Long.

Kits contain 2 liners & 1 outer stocking

- Liner ○
- Stocking ●

MADE-TO-MEASURE

Tailor-made to patient's individual dimensions.

Kits contain 2 liners & 1 outer stocking

- Liner ○
- Stocking ●●

We recommend the Altipress 40 Leg Ulcer kit as an **alternative treatment**, for independent patients with the will and ability to self care.

We also recommend it for healed patients who are **transitioning** out of bandages into preventative hosiery.



ALTI PRESS LINERS

Available separately.

Appropriate for:

- Prevention
- Maintenance therapy
- Palliative care
- Intolerance or non-concordance to higher levels of compression

BRITISH STANDARD COMPRESSION HOSIERY

Altiform compression hosiery is specifically designed to improve blood circulation in the legs. It provides graduated compression that gently squeezes the leg, pushing the blood back towards the heart and reducing fluid build-up at the ankle.

All Altiform products are latex-free.

We recommend that Altiform is worn to prevent venous disease progression:

NICE* recommends that compression hosiery is worn **after treatment** to prevent leg ulcer recurrence.

Additionally, we recommend hosiery is worn during treatment, as prevention on the non-ulcerated leg.


STYLES AVAILABLE



BELOW KNEE



THIGH LENGTH

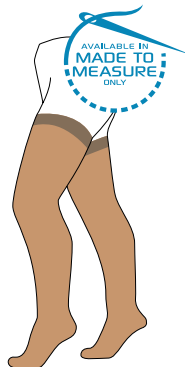

PLAIN HOLD UPS



HOLD UPS LACE

AVAILABLE IN:

STANDARD SIZES

Sizes: S, M, L, XL

MADE-TO-MEASURE

CLASS 1 (14-17 mmHg),

CLASS 2 (18 - 24 mmHg)

CLASS 3 (25 - 35 mmHg)

Black and Soft Beige ● ●

* NICE (2017), Leg ulcers - venous

EUROPEAN STANDARD COMPRESSION HOSIERY

Altiven compression hosiery is specifically designed for vascular conditions. It offers exceptional comfort with a soft, lightweight feel.

All Altiven products are latex-free.

AVAILABLE IN:

**7 STANDARD SIZES
MADE-TO-MEASURE**

EUROPEAN LOW COMPRESSION (14 - 18 mmHg),

CLASS 1 (18 - 21 mmHg),

CLASS 2 (23 - 32 mmHg)

CLASS 3 (34 - 46 mmHg)

Black and Soft Beige ● ●


STYLES AVAILABLE



BELOW KNEE



THIGH LENGTH

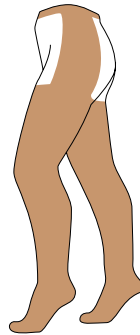


THIGH WITH WAIST ATTACHMENT



PLAIN HOLD UPS



TIGHTS

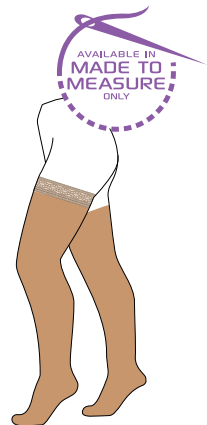

HOLD UPS LACE

AVAILABLE IN
MADE TO MEASURE
ONLY

COMPLETE COMPRESSION SOLUTIONS

TREATMENT

Urgo KTwo

**DUAL COMPRESSION SYSTEM
FOR TREATMENT OF ALL VENOUS AND
MIXED AETIOLOGY LEG ULCERS,
REGARDLESS OF PATIENT MOBILITY**



**DUAL COMPRESSION
SYSTEM**

BENEFITS



Consistent application of the recommended therapeutic pressure of ~40 mmHg¹ - due to unique pressure spot indicators



Provides **sustained compression** and stays in place for up to 7 days²



Optimal compression **for all patients, regardless of their mobility**²



Superior patient-rated comfort day and night - promoting compliance³



Read the UrgoKTwo information leaflet before use

COMPLETE COMPRESSION SOLUTIONS

ALTERNATIVE TREATMENT

Altipress[®] 40

LEG ULCER KIT

FOR TREATMENT AND PREVENTION OF VENOUS AND MIXED AETIOLOGY LEG ULCERS

AN ALTERNATIVE TREATMENT OPTION FOR SELF-CARING
PATIENTS AND THOSE INTOLERANT TO BANDAGES

BENEFITS

- Easy to apply with a unique elasticated toe and heel design
- Competitively priced
- British Skin Foundation accredited
- Made in the UK
- Latex-free
- Made-to-Measure range is manufactured within **72 hours**



COMPLETE COMPRESSION SOLUTIONS

PREVENTION

Altiform[®]

**BRITISH STANDARD HOSIERY
FOR PREVENTION OF VENOUS DISEASE
PROGRESSION AND MANAGEMENT OF
LOW-RISK CONDITIONS**

BENEFITS

- Easy to apply with a unique elasticated toe and heel design
- British Skin Foundation accredited
- Made in the UK
- Latex-free
- Made-to-Measure range is manufactured within **72 hours**



COMPLETE COMPRESSION SOLUTIONS

PREVENTION

Altiven[®]

**EUROPEAN STANDARD HOSIERY
FOR PREVENTION OF VENOUS DISEASE
PROGRESSION AND MANAGEMENT OF LOW-
RISK CONDITIONS**

BENEFITS

- Easy to apply with a unique elasticated toe and heel design
- Competitively priced
- British Skin Foundation accredited
- Made in the UK
- Latex-free
- Made-to-measure range is manufactured within **72 hours**





COMPRESSION TOP TIPS:

ASSESSMENT:

- ✓ Perform holistic assessment prior to applying compression
- ✓ Ensure ongoing, regular assessment

TREATMENT:

- ✓ Understand benefits of multi-component compression systems¹
- ✓ Encourage patients with healed ulcers to wear preventative hosiery.
- ✓ Ensure hosiery is worn as prevention on the second leg (non-ulcerated).

UNDERSTANDING THE PATIENT:

- ✓ Engage with the patient/ carer to ensure partnership and support
- ✓ Acknowledge physiological and psychological issues that come with leg ulcers
- ✓ Ensure the patient understands his/her own condition and the need for compression



COMPRESSION GARMENT RANGE



Altipress® Altiform® Altiven®



Accredits Urgo Medical research in
wound healing and healthy skin




ABOUT



FULL COMPRESSION PORTFOLIO

Urgo Medical, a market leader in advanced woundcare and compression, presents a full compression portfolio including dual-layer bandages (**UrgoKTwo**), four-layer bandages (**KFour**) and hosiery, for all your patient's needs, from treatment to prevention.

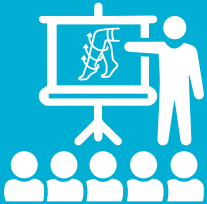

SUPPORT & TRAINING AT EVERY STEP

Urgo Medical regional teams are available to train, educate, hold study events and support the implementation of pathways.

- Hosiery study days, including CPD modules, practical sessions and product demonstrations
- Online resources for pharmacists
- Support designing, printing and delivering pathways.



**MADE IN
BRITAIN**

MADE IN THE UK

All our hosiery is manufactured in our factory in Shepshed, Loughborough. We offer site visits to demonstrate our commitment to the highest quality compression solutions.



FAST DELIVERY

We manufacture locally, resulting in market-leading delivery time.

Once your Made-to-Measure order is received, it is manufactured within 72 hours.

Altipress[®]

LEG ULCER KITS AND LINER PACKS

Altiform[®]

BRITISH STANDARD COMPRESSION HOSIERY

Altiven[®]

EUROPEAN STANDARD COMPRESSION HOSIERY

Sockaid[™]

HOSIERY APPLICATOR



The Altipress 40 Leg Ulcer Hosiery kit offers a practical and discreet alternative to bulky bandage systems. It is ideal for a patient with the will and ability to self-care.

Consists of 2 stockings that are layered on top of each other:

- Layer 1: Liner - Applies 10 mmHg at the ankle
- Layer 2: British Standard Class III stocking - Applies 30 mmHg at the ankle

Together, they provide 40 mmHg at the ankle, which is required for leg ulcer healing. Altipress Liners are also available separately.

BENEFITS OF ALTIPRESS:

- ✓ 15 Ready-to-wear sizes - options for all your patient's needs
- ✓ Easy to apply with a unique elasticated toe and heel design
- ✓ Ideal for self-caring patients
- ✓ British Skin Foundation Accredited - Demonstrating our commitment to quality products that are not harmful to the skin
- ✓ Latex Free

INDICATIONS

- Treatment and management of venous leg ulcers
- Prevention of venous leg ulcer recurrence
- Patients not suitable for or tolerant of compression bandages

CONTRAINDICATIONS

- Peripheral Vascular Disease (PVD), confirmed with an ABPI (Doppler) reading of less than 0.8
- Acute or unstable Heart Failure
- Ischaemic Heart Disease (IHD), also called Coronary Artery Disease
- Untreated Cellulitis or Phlebitis of the Lower Limb

Proceed with caution in those patients who have:

- Diabetes
- Rheumatoid Arthritis
- Heavily exuding or large wounds
- Irregular shaped limb

These patients may require referral to a specialist for advice and additional assessment. Close supervision may be necessary in line with local policies/pathways.

AVAILABLE IN:

STANDARD SIZES

Sizes: S, M, L, XL, XXL.

Lengths: Short, Regular, Long.

Kits contain 2 liners & 1 outer stocking

- Liner ○
- Stocking ●

MADE-TO-MEASURE

Tailor-made to patient's individual dimensions.

Kits contain 2 liners & 1 outer stocking

- Liner ○
- Stocking ●●



Altipress Liners are available separately. Altipress Liners provide light compression (10 mmHg at the ankle) for maintenance therapy, palliative care and intolerance and/or non-concordance to higher levels of compression. They can be layered to provide higher levels of compression (e.g. 2 liners provide 20 mmHg at the ankle) or worn under other compression stockings.

CONTRAINDICATIONS

If any existing arterial conditions exist, please seek advice from a healthcare professional before use.

AVAILABLE IN:

STANDARD SIZES

Sizes: S, M, L, XL, XXL

Lengths: Short, Regular, Long

Standard size liner packs contain 3 liners in either:

- White ○
- Soft Beige ●

MADE-TO-MEASURE

Tailor-made to patient's individual dimensions.

Made-to-measure liner packs contain 2 liners in:


- White ○

MEASURING AND FITTING GUIDE

	B: Calf	C: Ankle
Small	32 - 38	19 - 25.5
Medium	34 - 40	21 - 27.5
Large	36 - 42	23 - 29.5
X Large	38 - 45	25 - 31.5
XX Large	40.5 - 50	26.5 - 34

	Leg Length
Short	34 - 38
Regular	38 - 42
Long	42 - 46

Measurements in cms

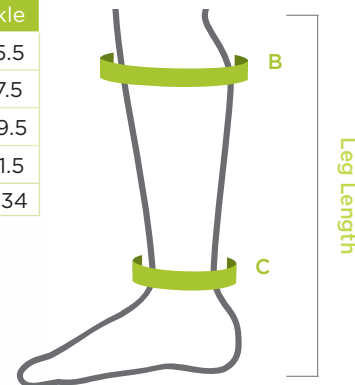

To order our Made-to-Measure hosiery, please request an order pad or order online at www.altimed.co.uk

Altiform compression hosiery is specially designed to improve blood circulation in the legs. It provides graduated compression that gently squeezes the leg, pushing the blood back towards the heart and reducing fluid build-up at the ankle.

Altiform is recommended to prevent venous disease progression.



BENEFITS:

- ✓ Discreet, comfortable and easy to wear
- ✓ Exceptional stretch, making it easy to slip over the heel
- ✓ Unique elasticated toe & heel for extra comfort
- ✓ Latex Free
- ✓ British Skin Foundation Accredited - Demonstrating our commitment to quality products that are not harmful to the skin
- ✓ Made in the UK.

INDICATIONS

Available in 3 Classes of Graduated Compression:

Class I – Light support

(14 – 17 mmHg compression at the ankle)

- Superficial or early varices
- Varicosis during pregnancy
- Swollen or aching legs & ankles
- Suitable for long distance travel

Class II – Medium support

(18 – 24 mmHg compression at the ankle)

- Varices of medium severity
- Venous leg ulcer treatment and prevention of recurrence
- Mild oedema
- Varicosis during pregnancy
- Suitable for long distance travel

Class III – Strong support

(25 – 35 mmHg compression at the ankle)

- Gross varices
- Post thrombotic venous insufficiency
- Gross oedema
- Venous leg ulcer treatment
- Prevention of recurrence

CONTRAINDICATIONS


- Peripheral Vascular Disease (PVD), confirmed with an ABPI (Doppler) reading of less than 0.8
- Acute or unstable Heart Failure
- Ischaemic Heart Disease (IHD), also called Coronary Artery Disease
- Untreated Cellulitis or Phlebitis of the Lower Limb

Proceed with caution in those patients who have:

- Diabetes
- Rheumatoid Arthritis

These patients may require referral to a specialist for advice and additional assessment. Close supervision may be necessary in line with local policies/pathways.


STYLES AVAILABLE:

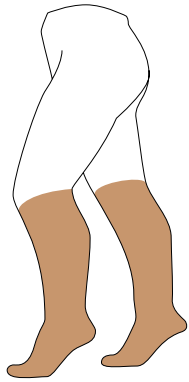

BELOW KNEE



THIGH LENGTH

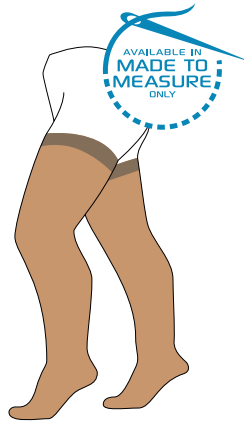


HOLD UPS PLAIN



HOLD UPS LACE



AVAILABLE IN STANDARD SIZES: SMALL, MEDIUM, LARGE, X-LARGE

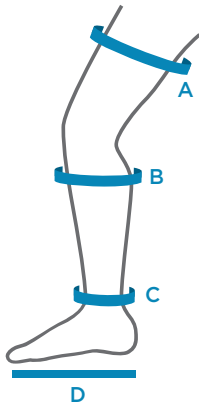

	A: Mid-thigh	B: Calf	C: Ankle	D: Foot
Small	41 - 54	32 - 38	19 - 25.5	20 - 26
Medium	44 - 58	34 - 40.5	21 - 27.5	22 - 28
Large	47 - 61	36 - 43.5	23 - 29.5	24 - 31
XLarge	50 - 64	38 - 47	25 - 32	26 - 33

Measurements in cms

	Below Knee		Thigh	
	Open Toe	Closed Toe	Open Toe	Closed Toe
Class I	●	●●	●	●●
Class II	●	●●	●	●●
Class III	●	●	●	●

Soft Beige ● Black ●

AND MADE-TO-MEASURE FOR A TAILORED FIT

Well-fitting compression stockings make all the difference to a patient's well-being and can improve concordance. Altiform Made-to-Measure stockings are made individually for each leg - giving the patient a precise, comfortable fit. Right and left leg measurements are often different - so different that sometimes two pairs of stock are needed to account for these differences. For this reason, Made-to-Measure stockings are a cost-effective option.

BENEFITS OF ALTIFORM MADE TO MEASURE HOSIERY:

- ✓ **Manufactured within 72 hours - ensuring fast delivery**
- ✓ Tailor-made to provide a unique and comfortable fit, to help improve concordance
- ✓ Easy to apply with a unique elasticated toe and heel design
- ✓ Unique patient identification number sewn into the sole for easy re-ordering
- ✓ Latex Free



To order our Made-to-Measure hosiery, please request an order pad or order online at www.altimed.co.uk


Altiven compression hosiery is specifically designed for vascular conditions, lymphoedema and oedema. It is available in 7 different sizes and Made-to-Measure for a tailored fit. It is also available in a variety of styles:



**BELOW
KNEE**



**THIGH
LENGTH**

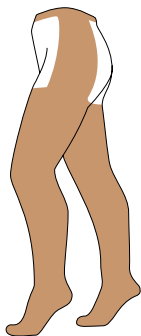


**THIGH WITH WAIST
ATTACHMENT**



**HOLD UPS
PLAIN**



TIGHTS

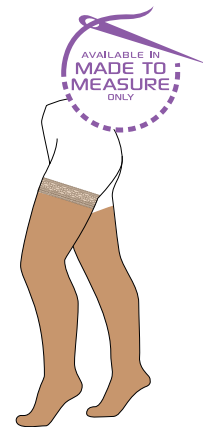

**HOLD UPS
LACE**

INDICATIONS

Low Compression (14 – 18 mmHg at the ankle)

Early onset and mild lymphoedema with minimal or no shape distortion. Stable condition. Elderly/arthritis patients. Pressure sensitive patients. ISL stages 0-II, maintenance, palliation, lipoedema, controlled cardiac oedema, dependency oedema, neurological deficit. May be considered for patients with ABPI 0.5-0.8 following full assessment. Prophylaxis.

European Class 1 (18 – 21 mmHg at the ankle)

Early onset and mild lymphoedema with minimal shape distortion. Lipoedema. Chronic Oedema. Stabilised condition. ISL stages 0-II, maintenance, palliation, lipoedema, controlled cardiac oedema, dependency oedema. ABPI >0.8.

European Class 2 (23 – 32 mmHg at the ankle)

Mild to Moderate lymphoedema, with minimal shape distortion. Maintenance after a leg ulcer has healed. Lipoedema. Long term maintenance. Varicose Veins. Venous conditions. ISL late stage II-III, phlebolymphoedema.

European Class 3 (34 – 46 mmHg at the ankle)

Moderate lymphoedema, with no shape distortion. To aid ulcer healing. Reduction therapy. Varicose Veins. Venous conditions. ISL stage III, phlebolymphoedema.



CONTRAINDICATIONS

- Peripheral Vascular Disease (PVD), confirmed with an ABPI (Doppler) reading of less than 0.8
- Acute or unstable Heart Failure
- Ischaemic Heart Disease (IHD), also called Coronary Artery Disease
- Untreated Cellulitis or Phlebitis of the Lower Limb


Proceed with caution in those patients who have:

- Diabetes
- Rheumatoid Arthritis

These patients may require referral to a specialist for advice and additional assessment. Close supervision may be necessary in line with local policies/pathways.

MEASURING AND FITTING GUIDE


Measurements in cms

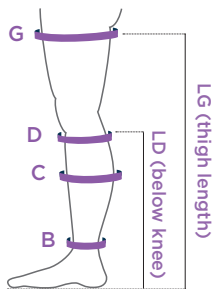

Size	I	II	III	IV	V	VI	VII
G Hold-up	43-48	45-52	49-56	53-60	56-64	60-68	64-72
G Hold-up Ex Wide	49-57	53-62	57-67	61-72	65-77	69-81	73-85
G	43-57	45-62	49-67	53-72	56-77	60-81	64-85
D	27-33	29-36	32-39	34-42	36-45	38-48	40-50
C	28-34	30-37	33-40	35-43	37-46	39-49	41-51
B	18-20	20-22	22-24	24-26	26-28	28-30	30-32
Thigh (LG)				Below Knee (LD)			
Regular				Regular			
Short				Short			

EUROPEAN STANDARD ARMSLEEVES

Altiven armsleeves are designed for the specific requirements of lymphoedema. Improved stretch makes the sleeve easier to put on. They are made specifically to each patient's measurements and provide optimum compression and graduation along the entire arm. Altiven offers exceptional comfort with a smooth, soft and lightweight feel.

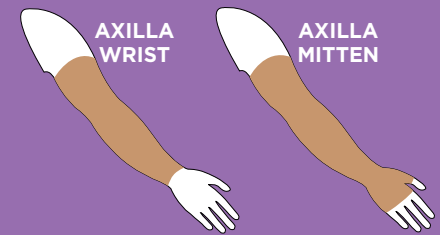

INDICATIONS

Low Compression

(12 - 16 mmHg)

Early onset and mild lymphoedema with minimal or no shape distortion. Stable condition. Elderly/arthritis patients. Pressure sensitive patients. Night maintenance. ISL stage I-II Prevention.



Class 1

(18 - 21 mmHg)

Mild lymphoedema with minimal shape distortion. Stable condition. Long term maintenance. ISL stage I-II.

Class 2

(23 - 32 mmHg)

Mild/Moderate lymphoedema, with minimal shape distortion. Long term maintenance. ISL stage II.



To order our Made-to-Measure hosiery, please request an order pad or order online at www.altimed.co.uk

Sockaid™

Developed by patients, Sockaid is designed to simplify sock, stocking and compression hosiery fitting.

BENEFITS OF A SOCKAID:

- ✓ Easy to use and easy to follow instructions
- ✓ Can be used in or out of bed
- ✓ Lightweight and robust

Available on FP10/GP10



ALTIPRESS 40 HOSIERY KITS

Kits comprise 1 Medical Stocking & 2 liners

SHORT LENGTH

MPC CODE	PIP CODE	SIZE	COLOUR
55SKO12	337-1325	S	●
55SKO22	337-1333	M	●
55SKO32	337-1341	L	●
55SKO42	337-1358	XL	●
55SKO52	337-1366	XXL	●

REGULAR LENGTH

MPC CODE	PIP CODE	SIZE	COLOUR
55RKO12	337-1374	S	●
55RKO22	337-1382	M	●
55RKO32	337-1390	L	●
55RKO42	337-1408	XL	●
55RKO52	337-1416	XXL	●

LONG LENGTH

MPC CODE	PIP CODE	SIZE	COLOUR
55LKO12	337-1424	S	●
55LKO22	337-1432	M	●
55LKO32	337-1440	L	●
55LKO42	337-1457	XL	●
55LKO52	337-1465	XXL	●

ALTIPRESS LINER PACKS

Liner Packs contain 3 liners

SHORT LENGTH

MPC CODE	PIP CODE	SIZE	COLOUR
55SLC10	337-1473	S	○
55SLC20	337-1481	M	○
55SLC30	337-1499	L	○
55SLC40	337-1507	XL	○
55SLC50	337-1515	XXL	○
55SLC12	341-8506	S	●
55SLC22	341-8696	M	●
55SLC32	341-8688	L	●
55SLC42	341-8670	XL	●
55SLC52	341-8613	XXL	●

REGULAR LENGTH

MPC CODE	PIP CODE	SIZE	COLOUR
55RLC10	337-1523	S	○
55RLC20	337-1531	M	○
55RLC30	337-1549	L	○
55RLC40	337-1556	XL	○
55RLC50	337-1564	XXL	○
55RLC12	341-8662	S	●
55RLC22	341-8654	M	●
55RLC32	341-8647	L	●
55RLC42	341-8621	XL	●
55RLC52	341-8605	XXL	●

LONG LENGTH

MPC CODE	PIP CODE	SIZE	COLOUR
55LLC10	337-1572	S	○
55LLC20	337-1580	M	○
55LLC30	337-1598	L	○
55LLC40	337-1606	XL	○
55LLC50	337-1614	XXL	○
55LLC12	341-8589	S	●
55LLC22	341-8563	M	●
55LLC32	341-8530	L	●
55LLC42	341-8522	XL	●
55LLC52	341-8514	XXL	●



To order our
Made-to-Measure hosiery,
request an order pad or order
online at www.altimed.co.uk

Altiform®

CLASS I. BELOW KNEE: Closed Toe

MPC CODE	PIP CODE	SIZE	COLOUR
511KC11	296-2611	S	●
511KC21	296-2629	M	●
511KC31	296-2637	L	●
511KC41	296-2645	XL	●
511KC12	296-2736	S	●
511KC22	296-2744	M	●
511KC32	296-2751	L	●
511KC42	296-2769	XL	●

CLASS I. BELOW KNEE: Open Toe

MPC CODE	PIP CODE	SIZE	COLOUR
511KO12	296-2694	S	●
511KO22	296-2702	M	●
511KO32	296-2710	L	●
511KO42	296-2710	XL	●

CLASS I. THIGH LENGTH: Closed Toe

MPC CODE	PIP CODE	SIZE	COLOUR
511TC11	296-2652	S	●
511KT21	296-2660	M	●
511KT31	296-2678	L	●
511KT41	296-2686	XL	●
511KT12	296-2819	S	●
511KT22	296-2827	M	●
511KT32	296-2835	L	●
511KT42	296-2843	XL	●

CLASS I. THIGH LENGTH: Open Toe

MPC CODE	PIP CODE	SIZE	COLOUR
511TO12	296-2777	S	●
511TO22	296-2785	M	●
511TO32	296-2793	L	●
511TO42	296-2801	XL	●

CLASS II. BELOW KNEE: Closed Toe

MPC CODE	PIP CODE	SIZE	COLOUR
512KC11	296-2892	S	●
512KC21	296-2900	M	●
512KC31	296-2918	L	●
512KC41	296-2926	XL	●
512KC12	296-3015	S	●
512KC22	296-3023	M	●
512KC32	296-3031	L	●
512KC42	296-3049	XL	●

CLASS II. BELOW KNEE: Open Toe

MPC CODE	PIP CODE	SIZE	COLOUR
512KO12	296-2975	S	●
512KO22	296-2983	M	●
512KO32	296-2991	L	●
512KO42	296-3007	XL	●

CLASS II. THIGH LENGTH: Closed Toe

MPC CODE	PIP CODE	SIZE	COLOUR
512TC11	296-2934	S	●
512TC21	296-2942	M	●
512TC31	296-2959	L	●
512TC41	296-2967	XL	●
512TC12	296-3098	S	●
512TC22	296-3106	M	●
512TC32	296-3114	L	●
512TC42	296-3122	XL	●

CLASS II. THIGH LENGTH: Open Toe

MPC CODE	PIP CODE	SIZE	COLOUR
512TO12	296-3056	S	●
512TO22	296-3064	M	●
512TO32	296-3072	L	●
512TO42	296-3080	XL	●

CLASS III. BELOW KNEE: Closed Toe

MPC CODE	PIP CODE	SIZE	COLOUR
513KC12	296-3338	S	●
513KC22	296-3346	M	●
513KC32	296-3353	L	●
513KC42	296-3361	XL	●

CLASS III. BELOW KNEE: Open Toe

MPC CODE	PIP CODE	SIZE	COLOUR
513KO12	296-3296	S	●
513KO22	296-3304	M	●
513KO32	296-3312	L	●
513KO42	296-3320	XL	●

CLASS III. THIGH LENGTH: Closed Toe

MPC CODE	PIP CODE	SIZE	COLOUR
513TC12	296-3411	S	●
513TC22	296-3429	M	●
513TC32	296-3437	L	●
513TC42	296-3445	XL	●

CLASS III. THIGH LENGTH: Open Toe

MPC CODE	PIP CODE	SIZE	COLOUR
513TO12	296-3379	S	●
513TO22	296-3387	M	●
513TO32	296-3395	L	●
513TO42	296-3403	XL	●



To order our
Made-to-Measure hosiery,
request an order pad or order
online at www.altimed.co.uk

CLASS 1. BELOW KNEE: Size I

CODE	TOE	LENGTH	COLOUR
541K1C1R	Closed	Regular	●
541K1C2R	Closed	Regular	●
541K1C1S	Closed	Short	●
541K1C2S	Closed	Short	●
541K1O1R	Open	Regular	●
541K1O2R	Open	Regular	●
541K1O1S	Open	Short	●
541K1O2S	Open	Short	●

CLASS 1. BELOW KNEE: Size II

541K2C1R	Closed	Regular	●
541K2C2R	Closed	Regular	●
541K2C1S	Closed	Short	●
541K2C2S	Closed	Short	●
541K2O1R	Open	Regular	●
541K2O2R	Open	Regular	●
541K2O1S	Open	Short	●
541K2O2S	Open	Short	●

CLASS 1. BELOW KNEE: Size III

541K3C1R	Closed	Regular	●
541K3C2R	Closed	Regular	●
541K3C1S	Closed	Short	●
541K3C2S	Closed	Short	●
541K3O1R	Open	Regular	●
541K3O2R	Open	Regular	●
541K3O1S	Open	Short	●
541K3O2S	Open	Short	●

CLASS 1. BELOW KNEE: Size IV

541K4C1R	Closed	Regular	●
541K4C2R	Closed	Regular	●
541K4C1S	Closed	Short	●
541K4C2S	Closed	Short	●
541K4O1R	Open	Regular	●
541K4O2R	Open	Regular	●
541K4O1S	Open	Short	●
541K4O2S	Open	Short	●

CLASS 1. BELOW KNEE: Size V

541K5C1R	Closed	Regular	●
541K5C2R	Closed	Regular	●
541K5C1S	Closed	Short	●
541K5C2S	Closed	Short	●
541K5O1R	Open	Regular	●
541K5O2R	Open	Regular	●
541K5O1S	Open	Short	●
541K5O2S	Open	Short	●

CLASS 1. BELOW KNEE: Size VI

541K6C1R	Closed	Regular	●
541K6C2R	Closed	Regular	●
541K6C1S	Closed	Short	●
541K6C2S	Closed	Short	●
541K6O1R	Open	Regular	●
541K6O2R	Open	Regular	●
541K6O1S	Open	Short	●
541K6O2S	Open	Short	●

CLASS 1. BELOW KNEE: Size VII

541K7C1R	Closed	Regular	●
541K7C2R	Closed	Regular	●
541K7C1S	Closed	Short	●
541K7C2S	Closed	Short	●
541K7O1R	Open	Regular	●
541K7O2R	Open	Regular	●
541K7O1S	Open	Short	●
541K7O2S	Open	Short	●

CLASS 2. BELOW KNEE: Size I

CODE	TOE	LENGTH	COLOUR
542K1C1R	Closed	Regular	●
542K1C2R	Closed	Regular	●
542K1C1S	Closed	Short	●
542K1C2S	Closed	Short	●
542K1O1R	Open	Regular	●
542K1O2R	Open	Regular	●
542K1O1S	Open	Short	●
542K1O2S	Open	Short	●

CLASS 2. BELOW KNEE: Size II

542K2C1R	Closed	Regular	●
542K2C2R	Closed	Regular	●
542K2C1S	Closed	Short	●
542K2C2S	Closed	Short	●
542K2O1R	Open	Regular	●
542K2O2R	Open	Regular	●
542K2O1S	Open	Short	●
542K2O2S	Open	Short	●

CLASS 2. BELOW KNEE: Size III

542K3C1R	Closed	Regular	●
542K3C2R	Closed	Regular	●
542K3C1S	Closed	Short	●
542K3C2S	Closed	Short	●
542K3O1R	Open	Regular	●
542K3O2R	Open	Regular	●
542K3O1S	Open	Short	●
542K3O2S	Open	Short	●

CLASS 2. BELOW KNEE: Size IV

542K4C1R	Closed	Regular	●
542K4C2R	Closed	Regular	●
542K4C1S	Closed	Short	●
542K4C2S	Closed	Short	●
542K4O1R	Open	Regular	●
542K4O2R	Open	Regular	●
542K4O1S	Open	Short	●
542K4O2S	Open	Short	●

CLASS 2. BELOW KNEE: Size V

542K5C1R	Closed	Regular	●
542K5C2R	Closed	Regular	●
542K5C1S	Closed	Short	●
542K5C2S	Closed	Short	●
542K5O1R	Open	Regular	●
542K5O2R	Open	Regular	●
542K5O1S	Open	Short	●
542K5O2S	Open	Short	●

CLASS 2. BELOW KNEE: Size VI

542K6C1R	Closed	Regular	●
542K6C2R	Closed	Regular	●
542K6C1S	Closed	Short	●
542K6C2S	Closed	Short	●
542K6O1R	Open	Regular	●
542K6O2R	Open	Regular	●
542K6O1S	Open	Short	●
542K6O2S	Open	Short	●

CLASS 2. BELOW KNEE: Size VII

542K7C1R	Closed	Regular	●
542K7C2R	Closed	Regular	●
542K7C1S	Closed	Short	●
542K7C2S	Closed	Short	●
542K7O1R	Open	Regular	●
542K7O2R	Open	Regular	●
542K7O1S	Open	Short	●
542K7O2S	Open	Short	●

CLASS 1. STANDARD HOLD-UPS: Size I

CODE	TOE	LENGTH	COLOUR
541E1C1R	Closed	Regular	●
541E1C2R	Closed	Regular	●
541E1O1R	Open	Regular	●
541E1O2R	Open	Regular	●
541E1C1S	Closed	Short	●
541E1C2S	Closed	Short	●
541E1O1S	Open	Short	●
541E1O2S	Open	Short	●

CLASS 1. STANDARD HOLD-UPS: Size II

541E2C1R	Closed	Regular	●
541E2C2R	Closed	Regular	●
541E2O1R	Open	Regular	●
541E2O2R	Open	Regular	●
541E2C1S	Closed	Short	●
541E2C2S	Closed	Short	●
541E2O1S	Open	Short	●
541E2O2S	Open	Short	●

CLASS 1. STANDARD HOLD-UPS: Size III

541E3C1R	Closed	Regular	●
541E3C2R	Closed	Regular	●
541E3O1R	Open	Regular	●
541E3O2R	Open	Regular	●
541E3C1S	Closed	Short	●
541E3C2S	Closed	Short	●
541E3O1S	Open	Short	●
541E3O2S	Open	Short	●

CLASS 1. STANDARD HOLD-UPS: Size IV

541E4C1R	Closed	Regular	●
541E4C2R	Closed	Regular	●
541E4O1R	Open	Regular	●
541E4O2R	Open	Regular	●
541E4C1S	Closed	Short	●
541E4C2S	Closed	Short	●
541E4O1S	Open	Short	●
541E4O2S	Open	Short	●

CLASS 1. STANDARD HOLD-UPS: Size V

541E5C1R	Closed	Regular	●
541E5C2R	Closed	Regular	●
541E5O1R	Open	Regular	●
541E5O2R	Open	Regular	●
541E5C1S	Closed	Short	●
541E5C2S	Closed	Short	●
541E5O1S	Open	Short	●
541E5O2S	Open	Short	●

CLASS 1. STANDARD HOLD-UPS: Size VI

541E6C1R	Closed	Regular	●
541E6C1S	Closed	Short	●
541E6O1R	Open	Regular	●
541E6O1S	Open	Short	●
541E6C2R	Closed	Regular	●
541E6C2S	Closed	Short	●
541E6O2R	Open	Regular	●
541E6O2S	Open	Short	●

CLASS 1. STANDARD HOLD-UPS: Size VII

541E7C1R	Closed	Regular	●
541E7C1S	Closed	Short	●
541E7O1R	Open	Regular	●
541E7O1S	Open	Short	●
541E7C2R	Closed	Regular	●
541E7C2S	Closed	Short	●
541E7O2R	Open	Regular	●
541E7O2S	Open	Short	●

CLASS 2. STANDARD HOLD-UPS: Size I

CODE	TOE	LENGTH	COLOUR
542E1C1R	Closed	Regular	●
542E1C1S	Closed	Short	●
542E1O1R	Open	Regular	●
542E1O1S	Open	Short	●
542E1C2S	Closed	Regular	●
542E1C2R	Closed	Short	●
542E1O2R	Open	Regular	●
542E1O2S	Open	Short	●

CLASS 2. STANDARD HOLD-UPS: Size II

542E2C1R	Closed	Regular	●
542E2C1S	Closed	Short	●
542E2O1R	Open	Regular	●
542E2O1S	Open	Short	●
542E2C2R	Closed	Regular	●
542E2C2S	Closed	Short	●
542E2O2R	Open	Regular	●
542E2O2S	Open	Short	●

CLASS 2. STANDARD HOLD-UPS: Size III

542E3C1R	Closed	Regular	●
542E3C1S	Closed	Short	●
542E3O1R	Open	Regular	●
542E3O1S	Open	Short	●
542E3C2R	Closed	Regular	●
542E3C2S	Closed	Short	●
542E3O2R	Open	Regular	●
542E3O2S	Open	Short	●

CLASS 2. STANDARD HOLD-UPS: Size IV

542E4C1R	Closed	Regular	●
542E4C1S	Closed	Short	●
542E4O1R	Open	Regular	●
542E4O1S	Open	Short	●
542E4C2R	Closed	Regular	●
542E4C2S	Closed	Short	●
542E4O2R	Open	Regular	●
542E4O2S	Open	Short	●

CLASS 2. STANDARD HOLD-UPS: Size V

542E5C1R	Closed	Regular	●
542E5C1S	Closed	Short	●
542E5O1R	Open	Regular	●
542E5O1S	Open	Short	●
542E5C2R	Closed	Regular	●
542E5C2S	Closed	Short	●
542E5O2R	Open	Regular	●
542E5O2S	Open	Short	●

CLASS 2. STANDARD HOLD-UPS: Size VI

542E6C1R	Closed	Regular	●
542E6C1S	Closed	Short	●
542E6O1R	Open	Regular	●
542E6O1S	Open	Short	●
542E6C2R	Closed	Regular	●
542E6C2S	Closed	Short	●
542E6O2R	Open	Regular	●
542E6O2S	Open	Short	●

CLASS 2. STANDARD HOLD-UPS: Size VII

542E7C1R	Closed	Regular	●
542E7C1S	Closed	Short	●
542E7O1R	Open	Regular	●
542E7O1S	Open	Short	●
542E7C2R	Closed	Regular	●
542E7C2S	Closed	Short	●
542E7O2R	Open	Regular	●
542E7O2S	Open	Short	●

 To order our Made-to-Measure hosiery, request an order pad or order online at www.altimed.co.uk

Sockaid™

MPC CODE	PIP CODE
60000	319-4016



To find out more please call us on:
01509 501720
 or visit us online at
www.urgo.co.uk www.altimed.co.uk

Altipress® Altiform® Altiven®



Accredits Urgo Medical research in wound healing and healthy skin

Urigo Limited - Sullington Road, Shepshed, Loughborough, LE12 9JG

*Manufactured and dispatched within 72 hours of receipt of the order. Notwithstanding forces beyond our control. Altiform, Altipress and Altiven are all registered trademarks of Urigo Limited



192560 November 2018