

ZIPZOC[®]

Zinc Oxide Medicated Stocking

**For the treatment of venous leg ulcers and
dermatological conditions**

ZIPZOC is a cotton medicated stocking impregnated with 20% Zinc Oxide which may be used as a primary contact layer to assist with the healing of wounds & skin conditions. ZIPZOC is preservative free, which means that there may be a reduced risk of reaction for those with sensitive skin. ZIPZOC helps to correct a local zinc deficit & applies anti-inflammatory & antibacterial properties which can aid healing¹.

- Quick and easy to apply
- Soothing & comfortable
- Preservative free formula
- Low adherent
- Cost effective care for up to 7 days
- Promotes patient self-care

**Contact
us today**
for more information or
to arrange a meeting
with our UK Clinical
Specialist.



Ordering Information

NHS Catalogue	Code
Zipzoc Stocking	EFA 029
FP10 Drug Tariff	PIP Code
Zipzoc Stocking (4's)	234-3739
Zipzoc Stocking (10's)	234-3747

EVLAN.

Evlan Pharma AB
www.evlan.se
E: pastebandages@evlan.se
T: +44 07554133321

1. Agren M (1990) Studies on Zinc in Wound Healing. Linköping University Medical Dissertations No. 320. Department of Pathology II, Faculty of Health Science. Linköping, Sweden.

ZIPZOC (zinc impregnated medicated stocking) **Abbreviated prescribing information.** Refer to summary of product characteristics before prescribing.
Composition: A sterile rayon stocking impregnated with an ointment containing 20% Zinc Oxide. Each stocking (80cm x 7cm) contains about 41.5g of ointment. **Therapeutic indications:** Aid to the treatment of chronic leg ulcers and other associated skin conditions. Where chronic venous insufficiency exists, the medicated stocking can be used as a primary contact layer under compression bandaging or hosiery. **Method of Administration:** Because ZIPZOC is preservative free, it must be changed no less frequently than weekly. For topical application, ZIPZOC should be applied to cover the lower leg from the base of the toes to below the knee. All folds should be smoothed out. To protect clothing, a suitable outer bandage should be worn. If chronic venous insufficiency exists, ZIPZOC may be used as a primary contact layer under compression therapy and may be left in situ for one week, before re-application.
Contra-indications: Arterial leg ulcers. Known hypersensitivity to zinc oxide or the excipients in the ointment. **Special Warnings and Special Precautions:** None known. **Interactions:** None known. **Pregnancy and Lactation:** There is no evidence to suggest ZIPZOC should not be used during pregnancy. **Undesirable Effects:** The skin of leg ulcer patients is easily sensitised to some topical medications. ZIPZOC contains no preservatives thereby reducing the risk of skin reactions. Reported reactions include, rash, erythema, itching and maceration of the wound edge. If the outer bandage is inappropriate or the medicated stocking is not changed with sufficient frequency it can cause the stocking to dry out. **Excipients:** Liquid Paraffin Ph. Eur. White Soft Paraffin BP. **Incompatibilities:** None reported. **Shelf Life:** 3 years. **Storage:** At or below 30°C. Keep all medicines out of the reach of children. **Packaging:** Polyethylene aluminium foil laminated pouches, each pouch containing a single medicated stocking. The outer carton will contain either 4 or 10 pouches. **Legal Category:** P **Market authorization number:** PL44616/0002 **Date of first authorisation/renewal of the authorisation:** 01 October 2006 **Date of revision of text:** 26 October 2018 **Trade Mark of Evlan Pharma AB** **Date of preparation:** October 2018